



# Raku-Ryu Cup Wide

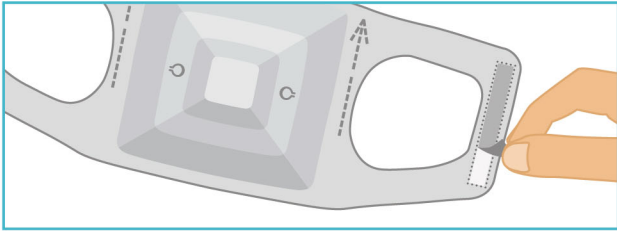

## STEP 1 Preparation

Peel the paper lining from the back side of the double-sided tape.

**CAUTION :** The paper lining cannot be flushed.

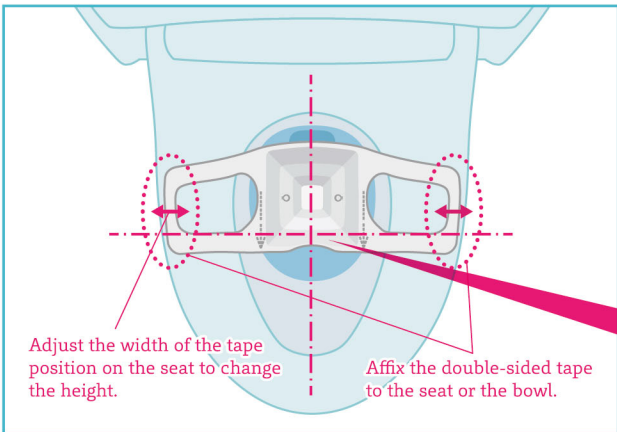


## STEP 2 Setting

Place the sampling sheet onto the back side of toilet seat or the bowl.

**CAUTION :** Place onto the toilet bowl if using toilet sheet cover. Property adjust the position after placement.

**Affix the double-sided tape to a little back than the center of the seat or the bowl.**


### Setting direction

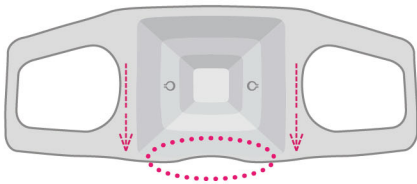

The notched area should face the front of the toilet.



### Setting image

Affix the double-sided tape to a little back than the center of the seat or the bowl.

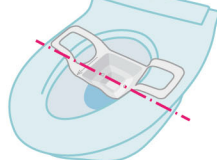

### Setting height

Between the middle height from adjusting point to pooled water.

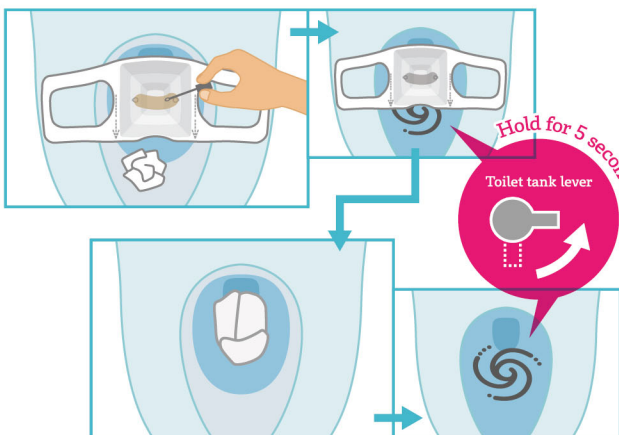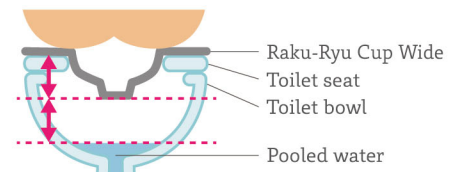

## STEP 3 Flush after stool sampling.

After toilet use, test the stool sample. At the first, flush the toilet paper accumulated in the toilet bowl using the water of amount of to flush the feces, then flush this product that feces are caught using the water of amount to flush the feces. **For tank-type toilets, flush after the tank is supplied with sufficient water.**

When flushing, in any case.

Please tilt the toilet tank lever enough to flush the stool. Please hold for 5 seconds as it is.